Embrace™ WetBond™ Restoration & PFM Repair Kit

PRODUCT DESCRIPTION

The Embrace WetBond Restoration & PFM Repair Kit contains Embrace First-Coat™, Embrace Opaquer, and Embrace Seal-n-Shine™, which are resin-based materials for priming, opaquing, and sealing fractured surfaces and repairs. It also contains Kool-Dam™ light cure liquid dam for protecting surfaces, and Porcelain Etch Gel.

XP-EMPFM-IN-07
REV: 07/2024

INSTRUCTIONS FOR USE

- Ceramic, metal and cured resin surfaces should be dry prior to applying Embrace resins. Tooth surfaces should be left slightly moist. Lightly dry and remove surface water with compressed air or a cotton pellet. Do not desiccate the tooth. Tooth surfaces should be shiny or glossy.
- Prepare surfaces for bonding:
 - Microabrade **metal** (and amalgam) surfaces.Rinse and dry well.
 - Apply a coat of **Kool-Dam** 1-2mm thick on soft tissue, tooth, or restorative surfaces requiring protection, and light cure.
 - Microabrade **ceramic** surfaces, and/or etch with **Pulpdent Porcelain Etch Gel**, carefully following instructions for use below. Rinse and dry well.
 - Etch previously cured **composite** surfaces with phosphoric acid etch gel for 10 seconds. Rinse and dry well.
 - If bonding to **dentin and enamel**, etch dentin 15 seconds and enamel 20 seconds with Etch-Rite 38% phosphoric acid gel. Apply DentTASTIC UNO or your preferred dental-enamel bonding agent in accordance with the manufacturer's instructions.
 - Apply a thin coat of **Embrace First-Coat** to the metal and/or ceramic surfaces with a flocked applicator tip. Light cure.
 - Apply **Embrace Opaquer**, if indicated, to metal or discolored surfaces using the brushes provided. Light cure.
 - Proceed with your preferred composite restorative or resin cement. Finish and polish.
 - Acid etch or microabrade the restored surface for 10 seconds. Rinse well. Dry ceramic, metal and cured resin surfaces. Lightly dry removing all surface water. Leave prepared tooth surface slightly moist.
 - Where necessary, place dental floss through the contacts and leave it in the embrasures.
 - Apply a thin coat of **Seal-n-Shine** to the restored surface with a flocked dispenser tip.
 - Lift dental floss through the contacts. **It is important for the contacts to be open.**
 - Light cure.

Porcelain Etch Gel Instructions for Use

- Protective eyewear must be worn by practitioner, assistant and patient. Protective gloves must be worn by practitioner and assistant.
- Place rubber dam and use Kool-Dam to isolate operative site and protect soft tissue.
- Assistant must stand ready at chairside and hold vacuum tip 2-3 inches from porcelain surface during the etching process to evacuate acid fumes.
- Securely attach 25-gauge luer lock applicator tip to syringe. Carefully start the flow of Porcelain Etch Gel on a pad and immediately dispose of this sample. Do not start flow in patient's mouth. Use a new applicator tip for each patient.
- Using gentle pressure, carefully apply Porcelain Etch Gel to porcelain surface and leave for one minute while evacuating fumes with vacuum. Some ceramic materials are resistant to etching. To achieve a surface suitable for bonding to these materials, air abrasion is recommended. Creating mechanical retention is advantageous. Cap syringe immediately after use.
- Remove gel with vacuum tip and rinse treated surface with copious amounts of water. Dry porcelain surface with clean uncontaminated air.

CAUTION

- Uncured resin materials may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves.
- Porcelain Etch Gel contains 9.6% hydrofluoric acid, a powerful and aggressive chemical that produces irritating fumes and must be handled with extreme care. Ingestion, contact with skin and eyes, and chronic inhalation of fumes are dangerous. Care must be taken to protect practitioner, assistant and patient. Pulpdent Porcelain Etch Gel has been buffered, gelled and packaged to minimize the dangers of hydrofluoric acid, provided it is used in accordance with these instructions.
- If Porcelain Etch Gel comes in contact with skin or soft tissue, rinse exposed area with copious amounts of water. Use high-speed evacuation system to remove water.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:
Pulpdent Corporation
80 Oakland Street, Watertown, MA 02472 USA
Tel: 617-926-6666 / Fax: 617-926-6262
email: pulpdent@pulpdent.com
Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Cenger Scandinavia A/S Urlevvej 6 B 8783 Hornslyd Denmark