## Dentin Conditioning Gel

### 22% Polyacrylic Acid High Molecular Weight

For removing the smear layer from dentin prior to bonding and cementation.

Pulpdent Dentin Conditioning Gel is a thixotropic 22% polyacrylic acid gel for removing the smear layer from dentin. The large molecular size of the polyacrylic acid occludes the dentinal tubules and does not penetrate to the pulp.

#### Caution:

Never apply Dentin Conditioning Gel directly on the pulp. If there is a pulpal exposure, vital pulp therapy using Pulpdent Paste or Multi-Cal is recommended.

#### Directions for Use:

1. Isolate the operative area to maintain a clean, dry field.
2. Securely attach a 22-gauge applicator tip to syringe. **Start flow on pad; not in patient's mouth.** Do not push on plunger with thumb. For safe, controlled dispensing, rest plunger in palm of hand and pull back on syringe barrel with thumb, index and middle fingers. Do not force plunger down into barrel. Use a new applicator tip for each patient.
3. Apply Dentin Conditioning Gel to dentin for 20 seconds.
4. Rinse with copious amounts of water.
5. Proceed with your preferred technique for dentin bonding or cementation.
6. Re-cap syringe immediately after use.

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**PULPDENT Corporation**
80 Oakland Street • Watertown, MA 02472 • U.S.A.