PULPDENT®


Etch-Royale™

Dental Etching Gel

Rx only

• 37% H₃PO₄

4 x 1.2 mL Etch-Royale

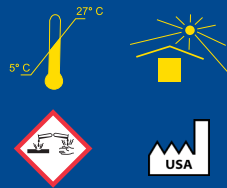PULPDENT Corporation • 80 Oakland Street • Watertown, MA 02472 • U.S.A.
www.pulpdent.com • pulpdent@pulpdent.com

PULPDENT®
Etch-Royale™
Dental Etching Gel

PULPDENT®

Etch Royale™ 37% PHOSPHORIC ACID Dental Etching Gel • For Acid Etching Dentin, Enamel and Glass Ionomer Cement

CE
0459

English

Etch Royale™

Dental Etching Gel – 37% Phosphoric Acid

Etch Royale is a thixotropic phosphoric acid etch gel that stays in place and will not run onto exposed tissue. Syringe dispensing through pre-bent small gauge applicator tips offers precise placement of gel even in relatively inaccessible areas. Etch Royale washes off easily with water.

PRODUCT DESCRIPTION

37% phosphoric acid dental etching gel

INDICATIONS

For acid etching dentin, enamel and glass ionomer cement

CONTRAINDICATIONS

Do not place on soft tissues

DIRECTIONS FOR USE

1. Prepare tooth and cleanse the enamel surface to be etched with a slurry of pumice or non-oil containing prophyl paste. Rinse and dry thoroughly.
2. Securely attach 25-gauge luer lock applicator tip to syringe. Start flow on pad; not in patient's mouth. Do not push on plunger with thumb. For safe, controlled dispensing, rest plunger in palm of hand and pull back on syringe barrel with thumb, index and middle fingers. Do not force plunger down into barrel.
3. Recommended etching time for enamel is 15-20 seconds and for dentin is 15 seconds. To the extent possible, apply Etch Royale first to enamel and then to dentin. Leave on dentin for 15 seconds. Cap syringe immediately after use.
4. Rinse off Etch Royale with copious amounts of water. Do not disturb this surface.

5. Leave surface wet, or dry surface with clean, uncontaminated air, depending on the instructions for use for the bonding system you are using.

Note: Etched enamel should appear chalky white. If not, reapply etch gel to enamel for 20 seconds and rinse with copious amounts of water.

WARNING:

If Etch Royale contacts skin or soft tissue, rinse off immediately with copious amounts of water.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to: Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

email: pulpdent@pulpdent.com

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Notification des incidents graves

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Hofmeister Dental B.V.
Lifco Dental AB
UNIDENT AB
My Trade Schweiz GmbH