PULPDENT®

EDTA 17% SOLUTION

EDTA 17% Solution is an effective chelating agent to assist in the instrumentation of root canals and in smear layer removal. EDTA helps decalcify the dentin in the canal wall. This makes it easier to remove dentin with files and reamers and helps the dentist enlarge and shape the canal for final filling with sealer and gutta percha.

PRODUCT DESCRIPTION

EDTA 17% Solution that has been buffered to a neutral pH

INDICATIONS

To assist with instrumentation and debridement during endodontic treatment

CONTRAINDICATIONS

Not a permanent root canal filling material

INSTRUCTIONS FOR USE

For Root Canal Cleaning and Shaping

1. Apply EDTA at entrance of canal with pellet or pipette, and file and ream canal in conventional manner.
2. Reapply EDTA as necessary until canal is enlarged to the desired size.
3. Rinse canal thoroughly so that all EDTA is removed from the canal.

For Smear Layer Removal Prior to Placement of Endodontic Posts

1. Prepare canal to receive a post.

2. Remove smear layer of dentinal debris from canal walls. Using a pipette, rinse canal with EDTA and immediately evacuate the EDTA with high suction for 10 seconds.
3. Rinse canal with water and remove excess water with air and/or paper points.

WARNING

Avoid contact with eyes.

STORAGE AND HANDLING

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.

REPORTING OF SERIOUS INCIDENTS

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation

80 Oakland Street, Watertown, MA 02472 USA

Tel: 617-926-6666 / Fax: 617-926-6262

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <https://www.fda.gov/medical-devices/dental-devices/multiple-use-dental-dispenser-devices>

Note: Refer to the Safety Data Sheet for a list of materials present in this product.

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Hofmeester Dental B.V.
Lifco Dental AB
UNIDENT AB
80 Oakland Street • Watertown, MA 02472 • U.S.A.