**EMBRACE™ VARNISH****PRODUCT DESCRIPTION**

EMBRACE™ Varnish is a resin-based varnish that provides sustained, time-release of fluoride, calcium and phosphate ions over a period of 4 hours or longer. The calcium and phosphate salts are coated with xylitol. Exposure to saliva dissolves the xylitol and releases the calcium and phosphate ions, which continuously react with fluoride ions in the saliva to form protective fluorapatite on the teeth.

EMBRACE™ Varnish contains 5% sodium fluoride, which is equivalent to 22,600 ppm fluoride (22.6 mg fluoride/mL). The material is tooth shade. It dries quickly forming a smooth thin film on the teeth.

**PRODUCT DESCRIPTION**

5% sodium fluoride varnish with Xylitol, calcium, and phosphate

**INDICATIONS**

EMBRACE™ Varnish treats hypersensitivity by releasing fluoride, calcium and phosphate ions that precipitate on and occlude dentinal tubules and fill superficial, non-cariouse enamel lesions.

**CONTRAINDICATIONS**

Ulcerative gingivitis and stomatitis

**CAUTION**

Limit ingestion by applying only as much varnish as necessary to cover tooth surfaces with a thin film. Adult patients should refrain from taking prescription fluoride supplements for a period of 24 hours following treatment, and children should refrain for 2-3 days.

**INSTRUCTIONS FOR USE**

1. Use back of cap to pierce seal on the tube.
2. Dry the teeth with a gauze pad. It is best if teeth are at least minimally clean, but it is not necessary to do a prophylaxis.
3. On a pad or mixing well, dispense 0.25 mL varnish for children and 0.4 mL for adults.
4. Using a brush, apply a very thin coat of varnish on the desired tooth surfaces. It is not necessary to use all the varnish.

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5. It is best if patients refrain from eating hard foods or drinking hot liquids or alcohol for 3-4 hours after treatment. Eating soft foods for the rest of the day is advantageous. Instruct patients not to brush their teeth for at least four hours.

6. EMBRACE Varnish can be removed by brushing and flossing.

**WARNING**

- EMBRACE Varnish may be irritating to respiratory tract if inhaled or to eyes or skin on contact. Varnish may be a sensitizer if exposed to prolonged or repeated contact. See SDS for further information.
- Dental practitioners should wear protective eyewear and gloves. Wash hands after use. Emergency eye wash fountain should be available in the office.
- EMBRACE Varnish contains no soy or gluten products. It contains a processed pine tree extract and not an extract of pine nut. Patients with allergies should consult their physician.

**STORAGE AND HANDLING**

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use if applying from a tube.

**REPORTING OF SERIOUS INCIDENTS**

Any serious incidents involving the device should be reported to the manufacturer and the Competent Authority of the Member State in which the incident occurred, or other appropriate regulatory body. Reports of serious incidents should be sent to:

Pulpdent Corporation  
 80 Oakland Street, Watertown, MA 02472 USA  
 Tel: 617-926-6666 / Fax: 617-926-6262  
 email: pulpdent@pulpdent.com

**Note:** Refer to the Safety Data Sheet for a list of materials present in this product.

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#### Hofmeester Dental B.V.
#### Lifco Dental AB
#### UNIDENT AB
80 Oakland Street • Watertown, MA 02472 • U.S.A.